# CLASSIC

## BETTAUFRICHTER

OVERBED GRAB RAIL | BEDBEUGELGREEP

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---

**Dear user,**

Thank you for choosing this product. This user manual describes how to prepare, operate, use, maintain and care for the product. It is a guideline for users and caregivers to ensure the safe and risk-free use. Please read this user manual carefully and keep it readily at hand at all times.

**CONTENTS**

The product is supplied in a cardboard box. The cardboard box serves for the future storage of the product and must not be discarded. Please first check whether the contents of the box is complete and undamaged. Notify the supplier immediately upon receipt should you detect any damage. Complaints raised at a later point in time cannot be accepted.

- Storage box
- Overbed grab rail
- This user manual

**SAFETY INSTRUCTIONS**

Ensure that the product always stands firmly. Only use the overbed grab rail on a level and firm floor. Avoid high lateral loads. When using the product beside a bed or adjustable armchairs, ensure that there is sufficient distance from moving parts. Before every use, make sure that the seams, the loop material and the fastening elements of the triangular handle are undamaged and securely fastened. For your own safety, replace these parts approximately every five years. Do not overload the overbed grab rail. Swinging from the standing aid is strictly forbidden. The overbed grab rail must not be combined with other products.

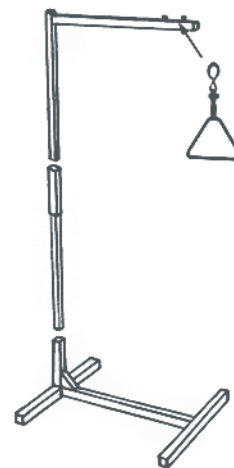
**INTENDED USE**

The overbed grab rail is only suitable for indoor use. It can be used in conjunction with most standard beds and, due to the low height of the foot, can also be used as a standing aid for seating furniture. Do not place the overbed grab rail to the side of the item of furniture, but always behind it at the head or back end. No additional fastenings on the wall or bed are required. The overbed grab rail can be dismantled easily and is therefore easy to transport and store. The maximum load/supported weight is 90 kg, making it suitable for a maximum body weight of 130 kg when used with a bed. If you do not intend to use the product for a long period of time, store it in a dry place away from dust and direct sunlight.

**ASSEMBLY AND USE**

Please observe the following instructions to install the overbed grab rail properly. We recommend that you have the

overbed grab rail installed and adjusted by your specialist dealer. Remove all parts from the packaging. Place the foot on the floor and insert the remaining pieces into each other (see illustration). Then attach the triangular handle and adjust the height of the triangular handle to the desired length. The product is now ready for use. Depending on the intended application, position the overbed grab rail behind the bed or armchair. Please always make sure that the arm with the triangular handle is located centrally above the patient, and that the patient does not place a lateral load on the overbed grab rail when using it. The height of the triangular handle is adjustable. Simply extend or shorten the loop.

**REUSE**

The overbed grab rail is designed to be reused after inspection by the specialist dealer, who can replace defective parts and disinfect the product if necessary.

**INDICATION**

Physical or medical impairment of the function of the pelvis, where patients still have upper body strength and the use of their arms. Supports independent sitting up in bed, for raising the buttocks (e.g. to use a bedpan) or moving into a wheelchair, toilet chair etc.

**CONTRAINDICATION**

If the overbed grab rail is used as intended, no contraindications are known.

**WORKING LIFE**

The expected working life of the overbed grab rail is 5 years provided it is used in accordance with this user manual and is undamaged. It can be used beyond this period of time, if it is in a safe condition and has not been damaged.

**CLEANING AND DISINFECTION**

All parts of the overbed grab rail should be cleaned and disinfected regularly. Clean with a damp cloth, using a mild cleaning agent for heavy soiling. For disinfection, please use a standard surface disinfectant, observing the manufacturer's instructions.

## DISPOSAL

Please dispose of the overbed grab rail in an environmentally friendly manner via an approved waste disposal company or your municipal waste disposal facility. Pay attention to current regulations.

## WARRANTY

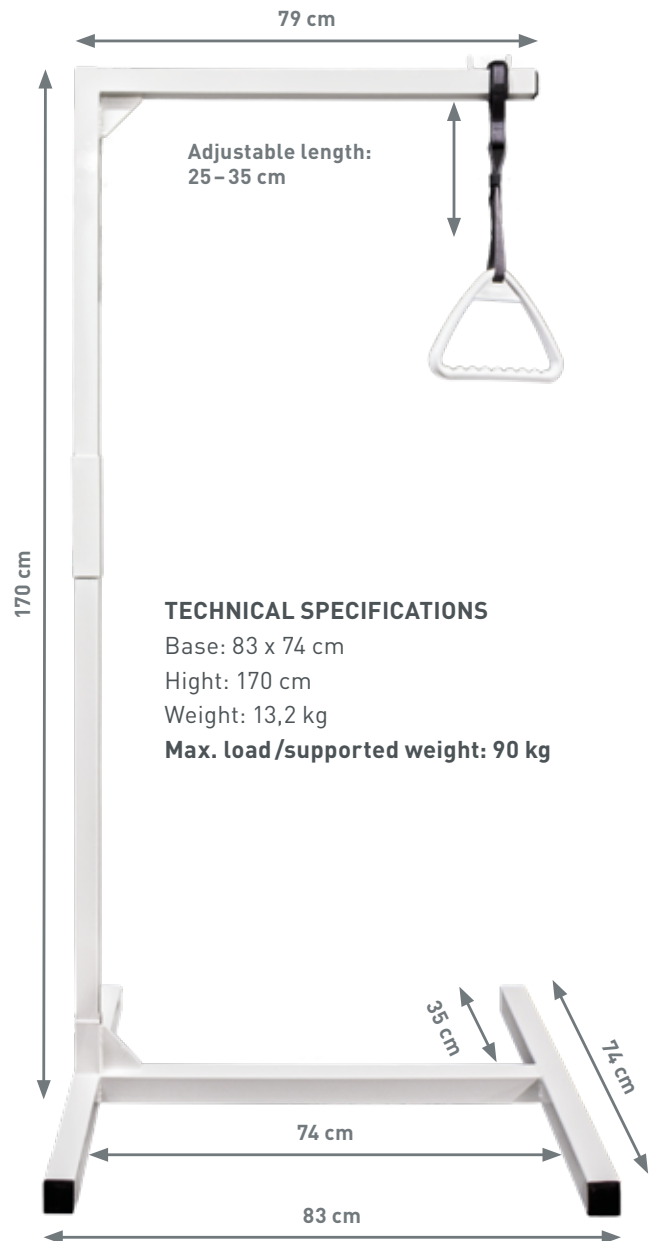
We offer a two-year warranty on manufacturing and material defects of the frame. Products subjected to improper use, negligence, poor care or poor storage conditions are not covered by the warranty. The warranty shall be void if any repairs or modifications are performed on the overbed grab rail or its components by unauthorised persons. In such cases, this shall preclude any product liability on the part of the manufacturer.

The manufacturer cannot accept any liability for damage to persons, property or animals in the following cases:

- Improper use of the device
- Use by unsuitable persons
- Incorrect installation of parts or accessories
- Modification or tampering without the prior consent of the manufacturer
- Failure to use original replacement parts or original accessories
- Non-compliance with the provisions of these operating and assembly instructions
- Exceptional occurrences

## MATERIAL

Frame and base: steel, powder-coated  
 Handle: plastic (PP)  
 Strap: polypropylene yarn  
 Clasp: plastic (POM)

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**CARELINE GmbH**
**CARELINE B.V.**
**CARELINE SAS**