AQUASAFE®

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Dear User,

Thank you for choosing this product. This user manual describes how to prepare, operate, use, maintain and care for the product. It is a guideline for users and caregivers to ensure the safe and risk-free use. Please read this user manual carefully and keep it readily at hand at all times.

INHALT

The product is supplied in a cardboard box. The cardboard box serves for the future storage of the product and must not be discarded. Please first check whether the contents of the box is complete and undamaged. Notify the supplier immediately upon receipt should you detect any damage. Complaints raised at a later point in time cannot be accepted.

Package content

- Storage box
- Toilet frame
- This manual

TECHNICAL SPECIFICATIONS

Item No. **52841**

Width: 60 cm

Depth: 45,5 cm

Height: 69,5–84,5 cm (7-way adjustable)

Width between armrests: 49 cm

Weight: 3 kg

Maximum load: 120 kg

Item No. **52842**

Width: 52,5–71 cm (7-way adjustable)

Depth: 42 cm

Height: 65–80 cm (7-way adjustable)

Width between armrests: 43,5–63 cm

Weight: 4 kg

Maximum load: 120 kg

USE

The stable and easy to clean toilet frame of the AQUASAFE® Series serves as a useful support when going to the toilet. The clearance under the strut is 42 cm (without height adjustment). It is placed around the toilet bowl and serves as a support when sitting up and sitting down.

ASSEMBLY [Item No. 52841]

The toilet frame is delivered fully assembled and is therefore ready for immediate use.

ASSEMBLY [Item No. 52842]

Attach the cross bars to the left and right frames. Push cross bar onto stub, align the holes and insert an 'e-clip'. Ensure both cross bars are set to the same width. **To adjust the width:** remove the 'e-clips' and slide the cross bar to the desired position. Align the holes and re-insert the 'e-clips'. Ensure both cross bars are set to the same width.

SAFETY INSTRUCTIONS

Regularly check the condition of the toilet frame. Pay attention to deformations and cracks. In this case, you must not use the toilet frame any longer. The rubber feet should be checked regularly and, if necessary, replaced. When adjusting the height, please make sure that all four legs touch the ground and are adjusted and locked at the same height. For Item No. 52842, also check regularly that all four clips are firmly engaged.

CLEANING AND DISINFECTION

All components of the toilet frame should be cleaned and disinfected regularly. Use a damp cloth for cleaning, and a mild detergent to remove any heavier soiling. Use a commercially available surface disinfectant. Please observe the manufacturer's instructions for use.

WARRANTY

The statutory warranty period of two years from date of purchase applies. The warranty covers any defects of the product which are demonstrably due to defects of material or manufacturing. Any damage incurred by natural wear and tear, intent, negligence, or improper operation or use is excluded from the warranty. The warranty also expires if unsuitable detergents are used. In the event of non-compliance with the instructions contained in this user manual, of improper maintenance or unauthorised modifications to the product without the manufacturer's prior written consent, all liability and warranty claims expire with immediate effect.

DISPOSAL

Please dispose of the toilet frame in an environmentally friendly manner via an approved waste disposal company or your local waste disposal facility. Please make sure to comply with any applicable regulations.

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