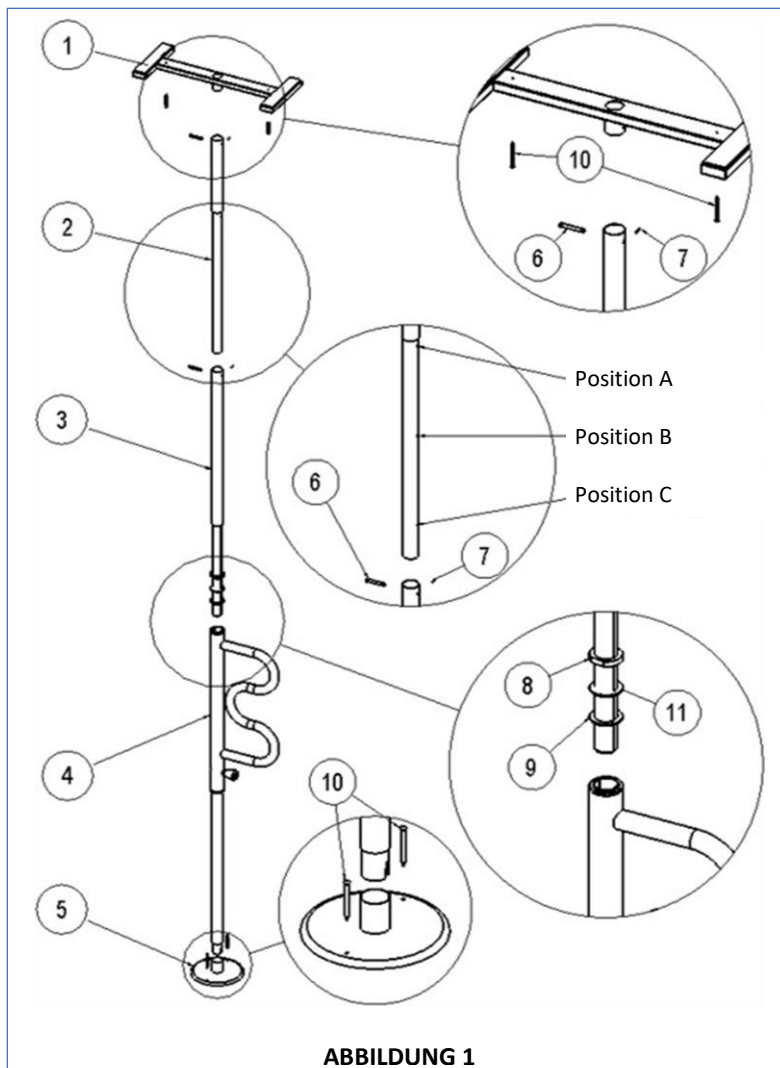
**Gebrauchsanweisung  
Instruction Manual  
Manuel d'instructions  
Instrucciones de uso  
Istruzioni per l'uso  
Instructiehandleiding**

**Boden-Deckenstange  
„Pole“**

Art.-Nr. / Product no. POLE1100

### 1.0 Vorwort:

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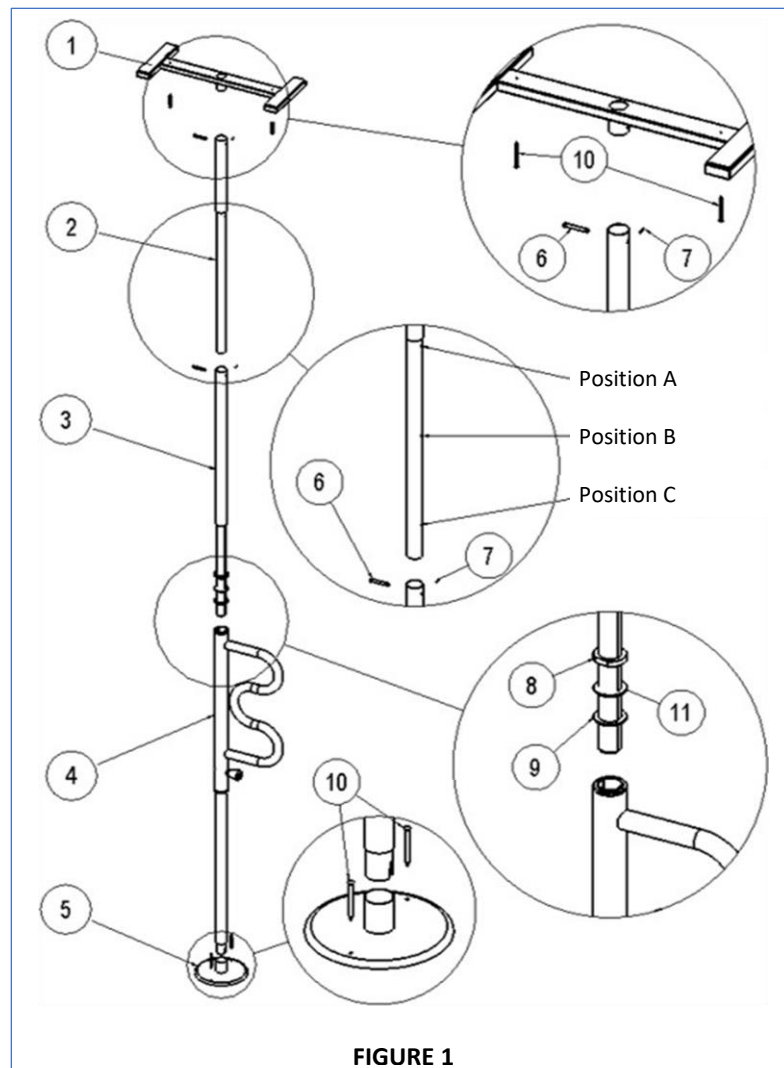
## 1.0 Preface:

Dear customer,

we are pleased that you have chosen a quality product from *rehastage GmbH*. A good relationship with our customers is of great importance to *rehastage*. Customer proximity means: fast service, as little bureaucracy as possible and close cooperation with our customers. Seek advice from your treating physician or therapist to ensure that this product is suitable for you and meets your needs.

**For your personal safety, please read these operating instructions carefully before using the product. If you have any problems understanding the information, warnings and instructions given here, please contact your specialist dealer to avoid possible injury or material damage due to incorrect operation.**

Do not throw away these instructions for use, even after the aid has been installed, so that you can use them later if necessary.

If you need spare parts or accessories, or simply have a question about your assistive device, we are here for you.

We want you to be satisfied with our products and our service. Therefore, we at *rehastage* are constantly working on the further development of our products. For this reason, changes in form, technology and equipment may occur in our product range. Therefore, no claims can be derived from the information and illustrations in these instructions for use.

**Instruction manual**  
**Floor-ceiling rod "Pole"**  
**Item number: POLE1100**

## 2.0 Intended Use:

The floor-ceiling bar "Pole" is intended as an upright and standing aid when using the washbasin and toilet, as well as an entry and exit aid when using the bathtub or shower.

The aim of the fitting is to enable independent or more independent personal hygiene or independent or more independent toilet use. Furthermore, the floor-ceiling bar "Pole" can also be used in other situations in which a corresponding support is needed when standing up or standing.

- Indication:

The floor-ceiling rod "Pole" serves patients

- with dysfunction of the lower extremities due to restricted or completely eliminated mobility of the joints,
- in case of limitation of muscular strength of the lower extremities,
- in the case of damage to the lumbar spine with impairment of joint mobility with sufficient functional capacity of the upper extremities and sufficient trunk stability
- in the case of generally reduced or impaired mobility without age restrictions.

- Contraindications:

⚠ Additional safety measures should be taken for persons belonging to certain risk groups. These include persons with a mental impairment (including persons under the influence of medication), very tall persons or persons who are very overweight, and any condition that impairs a person's ability to use the Floor Ceiling Bar "Pole" without assistance.

⚠ If you have an illness or injury, or are taking medication that affects your sense of balance or your ability to stand or sit, please consult your physician first before using this product. Improper use of the product may result in serious injury or damage.

⚠ The product is not intended to support the full weight of a person. This product is intended only as a balance aid and support when sitting and standing.

The floor ceiling bar "Pole" is a class I medical device according to Regulation (EU) 2017/745 of the European Parliament on medical devices.

## 3.0 Scope of delivery:

Part 1	Upper frame	1 piece
Part 2	Extension tube	1 piece
Part 3	Adjustment tube with threaded part	1 piece
Part 4	Stand tube with handle	1 piece
Part 5	Base plate	1 piece
Part 6	Bolts	2 pieces
Part 7	Split pin	2 pieces
Part 8	Adjusting nut	1 piece
Part 9	Washer	1 piece
Part 10	Wood screws 5 x 50	4 pieces
⚠	(Only for use in solid wood)	
Part 11	Plastic spacer	1 piece
Part 12	Mounting wrench	1 piece

## 4.0 Assembly instructions and operation

The floor-ceiling rod "Pole" is designed for ceiling heights from min. 210 to max. 300 cm.

Before starting the assembly, determine where you want to use this product and check whether the given ceiling height and the local conditions are suitable for the installation of the floor-ceiling rod.

Please note the following points!

⚠ When selecting the installation location, ensure that a minimum distance of 32 cm is maintained between the floor/ceiling rod/swivel handle and walls or fixed installations such as toilets, bathtubs and

**Instruction manual**  
**Floor-ceiling rod "Pole**  
**Item number: POLE1100**

shower trays, beds, etc. Falling short of this distance can, under certain circumstances, lead to the user becoming trapped and thus injured.

⚠ **Not suitable for clamp mounting of the floor-ceiling rod are:**

- Suspended ceilings,
- Ceilings in dry construction (e.g. plasterboard)
- Panelled ceilings made of wood, wood composite, plastic materials or similar materials

Due to the generally structurally weak pressure stability of these ceiling variants, it is not possible to achieve a sufficient clamping effect for a safe installation of the floor-ceiling rod.

Furthermore, the clamping process of the floor-ceiling rod can cause damage or permanent damage in the form of impressions, dents or breaks on or in the ceiling if the pressure stability of the ceiling construction is insufficient.

⚠ Furthermore, the floor-ceiling rod is NOT suitable for installation on sloping ceilings or on sloping levels. The rod must be installed perpendicular to the floor and ceiling. All protective rubber bumpers must be in contact with the ceiling and floor with evenly distributed pressure after installation.

⚠ After the complete assembly of the floor-ceiling rod at the installation site, it is mandatory to check the length of the exposed thread on the *adjustment tube with threaded part* (part 3). The measured length must not be less than 28 cm. Exceeding this maximum length leads to an unstable installation and thus to an increased risk of injury.

4.1 Remove all parts from the packaging and place them on a flat surface as shown in Figure 1.

4.2 Measure the ceiling height (distance between floor and ceiling) at the intended installation site and select the appropriate mounting form under 4.3.

4.3 **Ceiling height approx. 210 to max. 235 cm, the *extension tube* (part 2) is not mounted.**

Attach the *upper frame* (part 1) directly to the *adjustment tube with threaded part* (part 3) by aligning the holes in part 1 and part 3 and inserting the *bolt* (part 6) through the holes. After installing the *bolt* (part 6), insert the *cotter pin* (part 7) through the hole in the bolt and bend the end of the cotter pin so that the bolt is secured as shown in Figure 2.

**Ceiling height greater than 235 to max. 300 cm**

Attach the *top frame* (part 1) to the *extension tube* (part 2) by aligning the holes and fitting the *bolt* (part 6) and *cotter pin* (part 7) as shown in Figure 2.

As shown in Figure 1, the *extension tube* (part 2) has 3 perforations marked "A", "B" and "C":

- Perforation "A" is intended for ceiling heights from 235 to 260 cm,
- Perforation "B" is intended for ceiling heights from 260 to 285 cm and
- Perforation "C" is intended for ceiling heights from 285 to 300 cm.

Attach the *extension tube* (part 2) to the *threaded adjustment tube* (part 3) by aligning the holes marked "A", "B" or "C" on the *extension tube* (part 2) with the corresponding holes in the *threaded adjustment tube* (part 3) according to your ceiling height and secure with the *bolts* (part 6) and *cotter pins* (part 7) as shown in Figure 2.


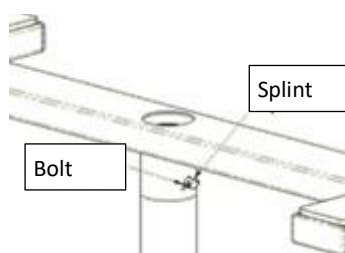
FIGURE 2

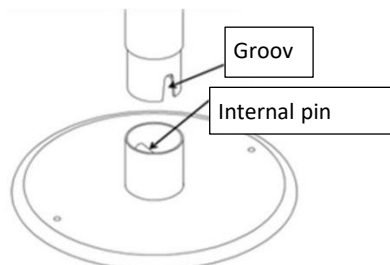**Instruction manual**  
**Floor-ceiling rod "Pole"**  
**Item number: POLE1100**

- 4.4 Attach the *base plate* (part 5) to the *standpipe with handle* (part 4) by aligning the groove in part 4 with the inside tenon in part 5 so that the groove is on both sides of the tenon to prevent rotation of the *base plate* (part 4) and *standpipe with handle* (part 5) (see Figure 3).

**FIGURE 3**

- 4.5 Insert the *threaded part from the adjustment tube* (part 3) into the *stand tube with handle* (part 4) and place the floor-ceiling rod "Pole" at the desired location, with the base plate part 5 on the floor and the upper frame (part 1) on the ceiling.

⚠ Make sure that the *plastic spacer washer* (part 11) is mounted between the *adjusting nut* (part 8) and the *washer* (part 9) -as shown in figure 1 before inserting the *adjusting tube with threaded part* (part 3) into the *standpipe with handle* (part 4).


- 4.6 To securely fasten the floor-ceiling rod "Pole" between the floor and the ceiling, turn the adjusting nut (part 8) clockwise using the supplied *installation wrench* (part 12). Tighten the lower *adjusting nut* until the floor-ceiling rod "Pole" is fixed between floor and ceiling.

⚠ Make sure that the floor/ceiling rod is aligned at a right angle (perpendicular) to the floor.

- 4.7 Additional securing:

If additional securing of the floor-ceiling rod "Pole" is to be done or is necessary, this can be done by screwing each 2 times through the *upper frame* (part 1) to the ceiling as well as through the *base plate* (part 5) to the floor of the installation environment as shown in Figure 4.

⚠ The use of suitable mounting material (not included in the scope of delivery). is absolutely necessary.

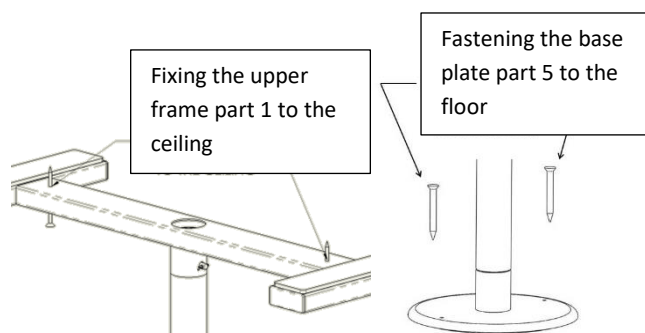**FIGURE 4**

- 4.8 **Operation:**

Pull the pin on the handle of the standpipe (part 4) and turn the curved handle to the desired position. Release the pin and continue to turn the handle until the spring-loaded pin locks into place, securing the handle in the desired position.

A slight play of the swivel handle is normal and is not a safety issue. This play allows the locking pin to easily engage in the 8 available user-friendly positions.

**Instruction manual**  
**Floor-ceiling rod "Pole"**  
**Item number: POLE1100**

**5.0 Cleaning Instructions:**

Clean the "Pole" floor ceiling rod regularly with a damp sponge, mild soap and a household disinfectant. Dry it with a dry soft cloth. Do not use abrasive cleaning agents such as scouring cream or powder, this can destroy or damage the surfaces and threads.

**6.0 Re-use / service life:**

Due to its simple and solid construction, the floor-ceiling rod "Pole" is suitable for re-use. The number of reuses depends on the condition of use of the device.

Before each reuse, the floor-ceiling rod "Pole" must be hygienically prepared and the technical and functional safety of the product checked and, if necessary, restored.

Assembly must be carried out as described under point 4.

If the product is used as intended in accordance with the specifications, the service life is expected to be 4 years. However, the effective life may vary depending on the frequency and intensity of use, as well as the care and general condition.

**7.0 Technical data:**

Suitable ceiling height:	210 - 300 cm
User weight:	max. 136 kg
Weight of product:	9.3 kg
Handle:	Swivel handle with 8 positions.
Material:	Galvanized tubular steel, partially powder coated

**8.0 Precautions / Warnings:**

**Please read the following instructions thoroughly before using the product.**

- ⚠ Check regularly that the floor-ceiling rod "Pole" is securely fastened between the floor and the ceiling.
- ⚠ Additional safety measures should be taken for persons belonging to certain risk groups. These include persons with a mental impairment (including persons under the influence of medication), very tall persons or persons who are very overweight, and any condition that impairs a person's ability to use the Floor Ceiling Bar "Pole" without assistance.
- ⚠ The handle on the floor-ceiling rod "Pole" is equipped with a pin with spring mechanism. This allows the handle to be turned and secured in various positions. Before using the product, make sure that the rotating handle is locked in a safe position. Failure to engage the handle may cause the user to fall, resulting in serious injury or damage.
- ⚠ If you have an illness or injury, or are taking medication that affects your sense of balance or your ability to stand or sit, please consult your physician first before using this product. Improper use of the product may result in serious injury or damage.
- ⚠ The product is not intended to support the full weight of a person. This product is intended only as a balance aid and support when sitting and standing.
- ⚠ The instructions accompanying the product must be followed carefully to ensure that the product is properly installed and assembled. Failure to follow the instructions may result in serious injury to the user(s) of the product.
- ⚠ Make sure that children do not play with or hang onto the product.
- **DISCLAIMER:** Neither Standers, Inc. nor the manufacturer/distributor shall be liable for any damage or injury that may result or has resulted from the improper use or installation of the product.
- In case of assembly problems, please contact the selling point (specialist trade) or directly to:  
rehastage GmbH  
Artlandstraße 80  
49610 Quakenbrück / Germany  
[gm@rehastage.de](mailto:gm@rehastage.de)
- In case of serious adverse events (deterioration of the user's condition) caused by the use of our product, we are obliged to report this to the manufacturer and the competent authority of the

**Instruction manual**  
**Floor-ceiling rod "Pole**  
**Item number: POLE1100**

respective Member State. Therefore, we ask you to inform us directly in such a case:

rehastage GmbH  
 Artlandstrasse 80  
 49610 Quakenbrück / Germany  
[gm@rehastage.de](mailto:gm@rehastage.de)

**9.0 Disposal / recycling of materials:**

If the aid was provided to you free of charge, it may not belong to you. If it is no longer needed, follow the return instructions provided by the organization that provided you with the device.

The following section describes the materials used in the device, including how to dispose of or recycle the device and its packaging. There may also be specific local regulations regarding disposal or recycling, these must be taken into account when disposing of the aid.

- Galvanized steel: Base body and attachments, clamping device, fastening screws
- Plastic: protective tube, anti-slip rubbers, bag for mounting material, packing material
- Carton: outer packaging

Disposal should be done through a disposal company or public disposal site.

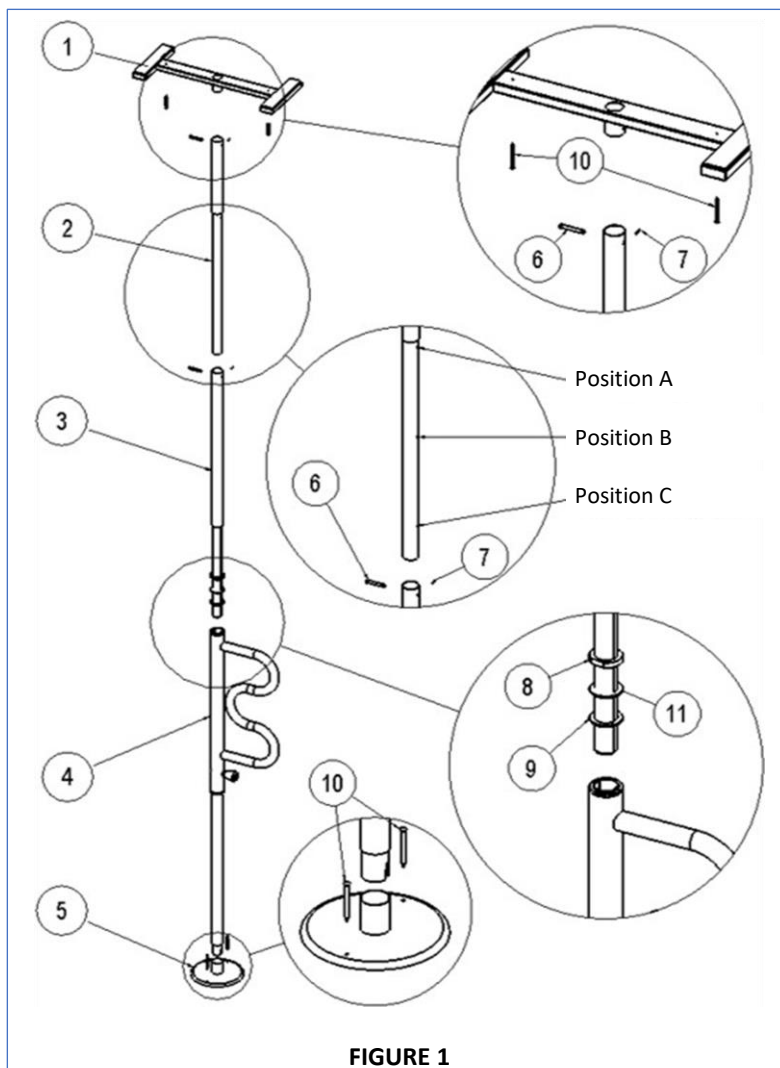
**10.0 Warranty:**

In addition to the statutory warranty, rehastage offers a warranty on this product for manufacturing or material defects of 2 years from the date of purchase. The warranty applies only to the first user of the item. Please keep the invoice, which is your proof of purchase. The warranty does not cover defects due to wear and tear, improper handling or commercial use of the item. The warranty is void if the item is repaired or constructively altered by anyone other than rehastage itself or a dealer authorized by rehastage. The warranty is void if repairs or modifications or replacement of individual parts that are not original rehastage spare parts are made by anyone other than the seller or supplying company. During the warranty period, rehastage shall decide whether to repair the device free of charge or replace it free of charge.

**11.0 Explanation of symbols used, markings and information to be provided:**

				
Manufacturer, name and address of manufacturer	Manufacturer's authorised representative within the European Community	Medical device	Attention! Information such as warnings or precautions	Follow the instructions for use!

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