VERMEIREN

377

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EN

Instructions for specialist dealer

This instruction manual is part and parcel of the product and must accompany every product sold.

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Version: A, 2017-01

All rights reserved, including translation.

No part of this manual may be reproduced in any form what so ever (print, photocopy, microfilm or any other process) without written permission of the publisher, or processed, duplicated or distributed by using electronic systems.

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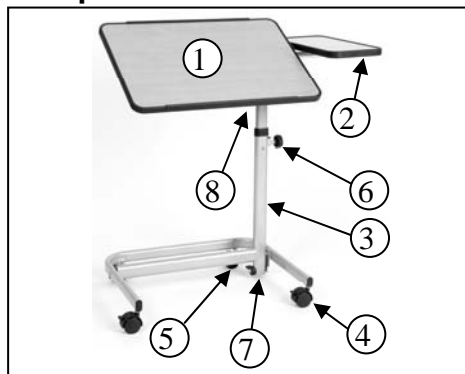
First of all we wish to thank you for the trust you placed in us by selecting a VERMEIREN product.

Before using the side table, read the instruction manual carefully: it will familiarise you with this product. You will also be given hints about the care and service of your side table. Please remember that observance of our hints would keep your side table in an excellent condition of preservation and it will function perfectly even after years of use.

If you have any further questions, please consult your specialist dealer.

Components

- 1 = Tray, large
- 2 = Tray, small
- 3 = Lower frame
- 4 = Wheels with brakes
- 5 = Wheels without brakes
- 6 = Star button height adjustment
- 7 = Lever angle adjustment frame
- 8 = Lever angle adjustment large tray

Technical details

Height	650 - 940 mm
Length	810 mm
Width	430 mm
Dimensions folded/dismantled	610 mm x 130 mm x 850 mm
Angle frame	0° - 75° - 90°
Angle large tray	-45° / -30° / -15° / 0° / 15° / 30° / 45°
Height lower frame	100 mm
Free height above the floor	70 mm Below vertical tube: 15 mm
Shelf area, small	190 x 410 mm (L x B)
Shelf area, large	610 x 410 mm (L x B)
Load small tray	max. 10 kg
Load large tray	max. 10 kg
Weight	8,35 kg
Diameter wheels	50 mm

Contents of consignment

- Lower-frame (including 2 castors with brake and 2 castors without brake)
- Upper frame (incl. adjustable tray)
- Instruction manual

Purpose

This side table can be pushed in beneath a patient's bed and used as a tray for holding various objects. It cannot be used for supporting the patient or other persons. It can also not be used for conveying persons or objects. The manufacturer does not accept liability for damage caused by improper handling.

Assembling

Push the lever ① downwards and move the tube ② of the lower frame upwards (75° or 90°). Check that the tube is firmly secured. Mount the large tray in the lower frame and secure it with the star knob ③ at the desired height (**the maximum height is until the locking pin ④ clicks in the hole**).

Mount the small tray with the washer and screw.

→ **N.B:** The levers, star knob must be tightened before any objects can be placed on the trays.

→ **N.B:** Check that the inserted tube is not jammed or skew and that they is pushed in fully.

Operating the brakes

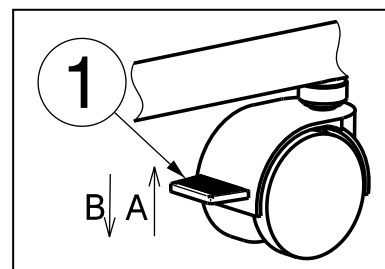
⚠ WARNING: The brakes can wear – Check the operation of the brakes before each use.

A: Engaging the brakes

With your foot, press the brake plate downwards till it locks in place. The wheels can now neither roll nor turn.


B: Disengaging the brakes

With your foot, press the brake plate upwards. The wheels have now been released and can both roll and swivel.

To put the table in a parking position, the two wheels must be locked in place.

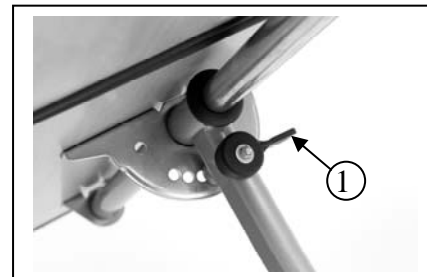Adjustments

Pull the lever ① downwards. Now the angle of the tray can be adjusted. Loosen the lever ①. Check that the tray is firmly secured.

→ **The stability of the product is affected when the tray is adjusted (danger of tipping over).**

→ The tray should be tilted only so far that nothing will fall off.

Safety

→ Do not stand on the tubes of the lower frame, since the table could tip over -- danger of injury !

→ Do not lean against or on the trays -- danger of tilting !

→ Do not lift the table by means of the trays. These parts are adjustable and might cause injury or damage when raised up if they are not properly secured.

→ Do not place any objects larger than the trays on them. These could fall and cause injury / damage.

→ Do not exceed the bearing capacity of the table (10 kg) with heavy objects -- danger of tipping over !

→ Do not convey persons or objects -- danger of injury !

→ The side table must only be set up indoors on level surfaces. When in use, check that all 4 castors press evenly on the floor.

→ Repair work may only be undertaken by the authorised dealer.

→ Take note of the instructions for care and service. The manufacturer is not liable for damage caused by improper servicing/care.

Care

It is recommended to use a soapy solution for cleaning. The cleaning liquid should in any case have a pH value of at least 6. Do not use a cleaning agent which contains solvents. Observe the instructions of the cleaning liquid used. Use a damp cloth and lukewarm water for cleaning. Avoid soaking. Do not use metal brushes or any other sharp-edged or pointed cleaning utensils. In the case of damage to the varnish or the woodwork, consult your specialist dealer. Regular care would ensure that your side table is preserved in a perfectly functional condition.

Disinfection

For sterilisation use only disinfectants suitable for treating varnished metal and wood surfaces. Observe the instructions of the cleaning liquid used. Use only proved processes (for scrubbing disinfection) and disinfectants included in the list of the Robert Koch Institute (information can be obtained from: www.rki.de). Disinfections may only be undertaken by trained persons. Consult your specialist dealer.

Repairs / Service / Inspection

You should use only genuine VERMEIREN replacement parts. Repairs and renovations may only be undertaken by trained persons. Make use of the services offered by your specialist dealer. He would gladly assist you further in regard to service and repairs.

Before you resume use of the table, it should be serviced according to the following criteria:

- Completeness
- Adjustment functions
- Check the castors (stability, tracking, fixing screws)
- Setting of the levers (Condition, thread, holding security)
- Frames, (deformation, stability)
- Trays, (closed surfaces, splinters, tears)
- Damaged varnish (remove possible rust and repaint)

Storage

between + 5° and + 45° Celsius
20% - 75% relative humidity

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ENGLISH

CONTRACTUAL WARRANTY

We offer 5 years of warranty on standard wheelchairs, lightweight wheelchairs 4 years. Electronic wheelchairs, tricycles and other products: 2 years (batteries 6 months) and multiposition wheelchairs 3 years. This warranty is limited to the replacement of defective or spare parts.

APPLICATION CONDITIONS

In order to claim its warranty, part "B" of this card has to be given to your official Vermeiren dealer. The warranty is only valid when parts are replaced by Vermeiren in Belgium.

EXCEPTIONS

This warranty is not valid in case of:

- damage due to incorrect usage of the wheelchair,
- damage during transport,
- involvement in an accident,
- a dismount, modification or repair carried outside of our company and/or official Vermeiren dealership,
- normal wear of the wheelchair,
- non-return of the warranty card

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- please return within 8 days of date of purchase or register your product at our website, <http://www.vermeiren.be/registration>

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Vermeiren Suisse S.A.